ColsigenreFresh™

5 single-use masks, 25ml each
 5 Masken zur einmaligen Verwendung, je 25 ml
 5 masques à usage unique, 25 ml chacun
 5 maschere monouso da 25 ml ciascuna

VEGAN




FRAGRANCE-FREE

Scan me!

colsigen.com

Directions for use / Anwendung /
 Mode d'emploi / Istruzioni per l'uso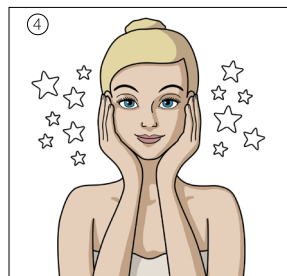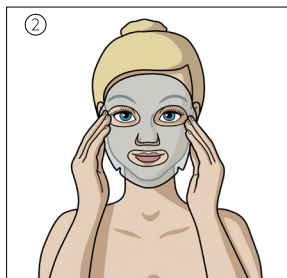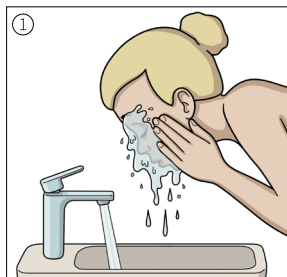

MD	Medical device Medizinprodukt Dispositif médical Dispositivo medico
	Consult instructions for use Gebrauchsanweisung beachten Consultez le mode d'emploi Consultare le istruzioni per l'uso
	Keep out of the reach of children Für Kinder unzugänglich aufbewahren A conserver hors de portée des enfants Tenere fuori della portata dei bambini
	Use by date Verfallsdatum Date limite d'utilisation Data di scadenza
	Do not use if the package is damaged Nicht verwenden, wenn die Verpackung beschädigt ist Ne pas utiliser si l'emballage est endommagé Non utilizzare se la confezione è danneggiata
	Do not re-use Nicht wiederverwenden Ne pas réutiliser Non riutilizzare
LOT	Batch code Seriennummer Numéro de lot Codice del lotto
	Date of manufacture Herstellungsdatum Date de fabrication Data di produzione
	Stratpharma AG, Aeschenvorstadt 57, CH-4051 Basel, Switzerland / Schweiz / Suisse / Svizzera
Date of preparation of this information / Ausstellungsdatum dieser Information / Date de préparation de cette information / Data di revisione del testo: 06/2025	

Stratpharma Switzerland
 Heal Better™

CF000INT1PI01-0225

[EN] Consumer information

Colsigen Refresh is a restorative face mask formulated with Humanized Type III Collagen.

Colsigen Refresh soothes, hydrates, and rejuvenates sensitive or compromised skin, repairing the damage from daily environmental stresses.

How Colsigen Refresh works?

This mask, designed with advanced fabric technology to conform to facial contours, helping to create a barrier that moisturizes evenly and ensures comfort during use.

Colsigen Refresh is designed for daily hydration and care of the skin, the mask supports the skin's natural moisture barrier, enhancing skin hydration and repair.

Colsigen Refresh is gentle enough for everyday use.

Intended purpose (intended use)

Colsigen Refresh is a transient protective barrier, which improves hydration and supports skin repair.

Indications

Intended to be used daily or as needed on all types of skin.

Indicated for dry, irritated and compromised skin.

Intended users

Colsigen Refresh is indicated for adults, children, and people with sensitive skin.

Contra-indications

Not suitable for individuals allergic to any of the ingredients.

Not suitable for use on open wounds or burns.

Warnings and precautions

- For external use only.
- Avoid contact with the eyes.
- If irritation occurs, discontinue use and consult your physician.
- Do not use if the package is damaged.
- Do not use after the expiration (EXP) date printed on the sachet.
- Within the European Union, serious incidents related to the use of Colsigen Refresh should be reported to Stratpharma and the respective competent authority in the country where the user is established.

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